


# GBR KIT (ONGBRK) RENEWAL 2021.10

• KIT composed of all necessary tools for GBR procedure including bone screw and bone tack

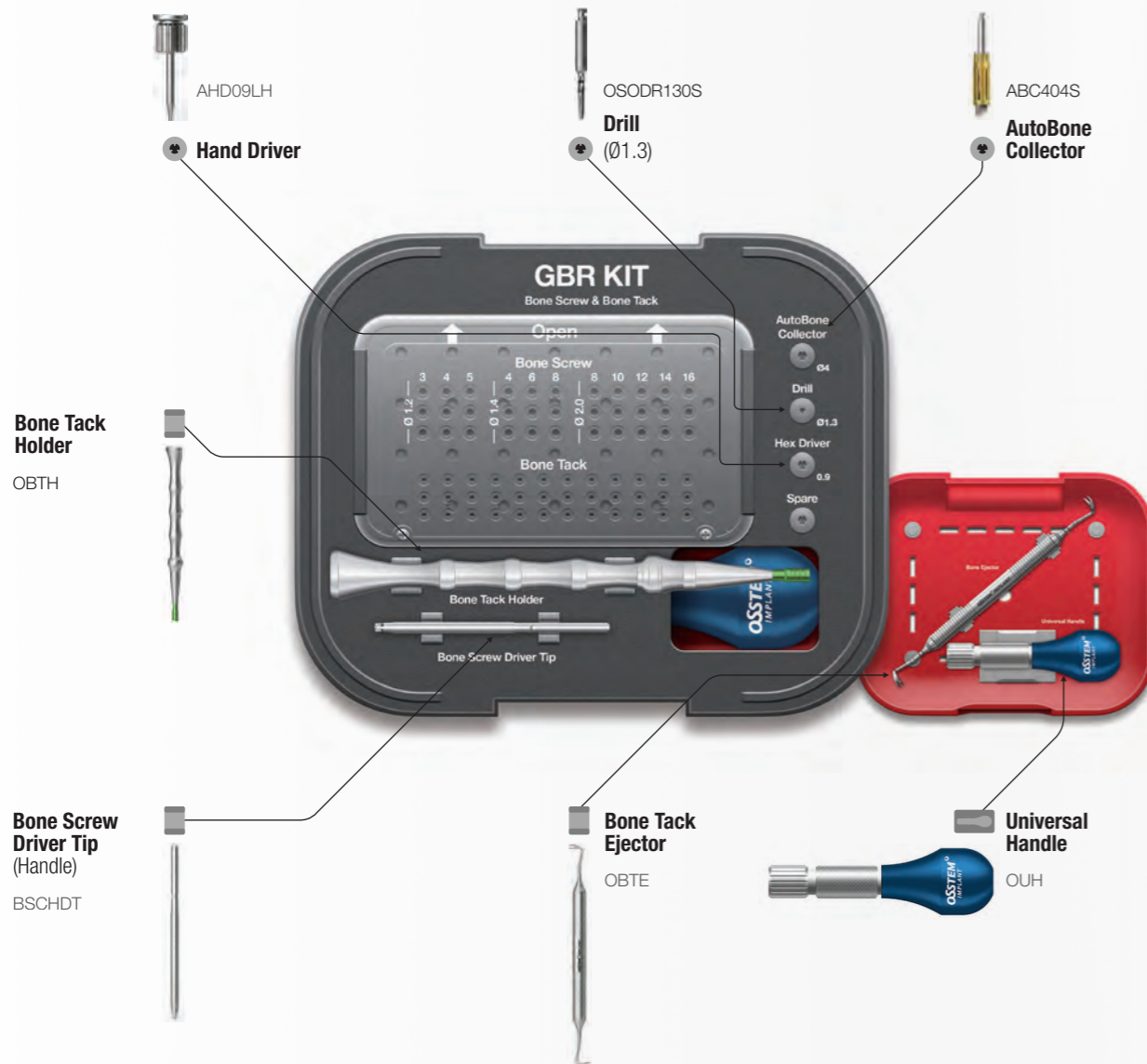

# GBR KIT Surgical Instruments - Bone Screw For Bone Screw Only

## Bone Screw Driver Tip (Handle)

- Used by connecting to Universal Handle
- Tighten the bone screw upright fully before use



**Cross**

BSCHDT

## Universal Handle

- Used by connecting to a bone screw driver tip (handle)



OUH

## Bone Screw Driver (Engine)

- Used by connecting with an engine
- Tighten the bone screw upright fully before use




**Cross**

BSCMD

## Ø1.3 Drill

- Used with a Ø 2.0 bone screw
- Recommended drilling speed : 800rpm

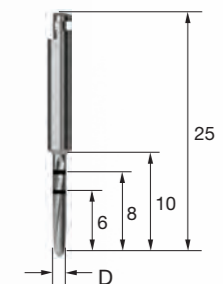

**D**

Ø1.3

OSODR130S

# GBR KIT Surgical Instruments - Bone Tack

## Bone Tack Holder **RENEWAL 2021.10**

- Used for holding the bone tack
- Design for stable grasping
- Anti-rotation feature
- Store after connecting to a cap in place



Bone Tack Holder



Bone Tack Holder Protector



Bone Tack Holder Tip




Bone Tack Holder Handle

## Bone Tack Ejector **2018.10**

- Used for removing the bone tack
- Leverage-based design for easy removal

※ If the bone tack covered with bone prevents the ejector from being inserted, use a 0.9 hex driver to reverse and lift the bone tack before using the ejector tack

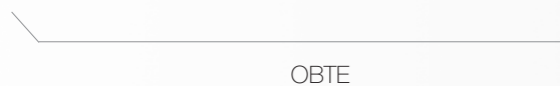

**OSSTEM**<sup>®</sup>  
IMPLANT